



Final lens:

OD: 7.60 (44.50D) -3.00 / 2.00 9.6
Prestige Reversed Optimum Extra™ Perfecta
VAOD: 0.8 (6/7.5 or 20/25) Near 0.65 (J4 or N5)

OS: 7.70 (43.75D) -5.00 / 2.50 9.6
Prestige Optimum Extra™ Perfecta
VAOS: 1.0 (6/6 or 20/20) Near 0.5 (J5 or N6)

BCVA OU: 1.0+ (6/6+ or 20/20+) Near 0.8 (J2 or N4)*

* the most important thing to note is the improved near vision

Discussion/conclusion: If it is not possible to obtain the desired near vision results with multifocal lenses in both eyes, then it is sometimes possible to fit a modified monovision. With this system, it is preferable to put distance vision in the dominant eye, and therefore to fit a Centre Distance lens in this eye. Then a Centre Near lens is fitted in the non-dominant eye. In order to improve the stability of the Centre Near lens, and to prevent the lens riding too high in the reading position, it is fit steeper. It is important to maintain good tear pump action. Correct edge lift and material choice are essential.

Lenses distributed by:



UCO
 Hoekvaartweg 21
 1771 RP Wieringerwerf
 Netherlands
www.ucolenz.nl

Telephone 0227-601837
 Fax 0227-602978
 e-mail info@ucolenz.nl

CONTAMAC

Contamac Ltd
 Orchard House Bearwalden Business Park
 Wendens Ambo Saffron Walden
 Essex CB11 4JX UK

Tel: +44 (0)1799 544400
 Fax: +44 (0)1799 543000

E-mail: professionalservices@contamac.com

CONTAMAC US

Contamac US
 806 Kimball Avenue
 Grand Junction, CO 81501

Tel: 1-(970) 242 3669 Fax: 1-(970) 243 5501
 Toll Free: 1-866 US CONTAMAC
 (1-866 872 6682)

E-mail: professionalservices@contamac.com